3SHAPE

---

EXOCAD

---

DENTAL WINGS

# DESS

# LIBRARY

# INSTALLATION

FOR HYBRID ABUTMENTS, CROWNS, BRIDGES AND BARS

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INS-12-MUL-02

18-10-2016

# 3Shape DESS ABUTMENTS LIBRARY\_[Version]

## ENGLISH

- 1 • Unzip the folder "3Shape DESS ABUTMENTS LIBRARY\_[Version].zip". This folder contains the file "3Shape DESS ABUTMENTS LIBRARY\_[Version].dme", the components of library DESS for 3Shape.
- 2 • Open the Dental System Control Panel and go to "Tools" → "Import/Export".
- 3 • Choose "Import materials". Windows Explorer opens and you need to choose the unzipped .dme file.
- 4 • The DESS Library is ready to be used.

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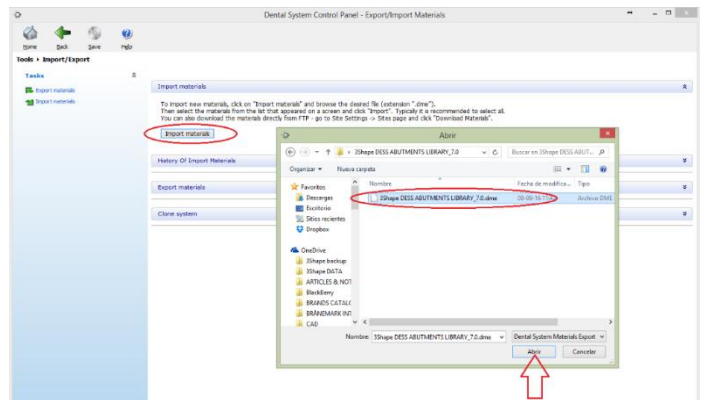
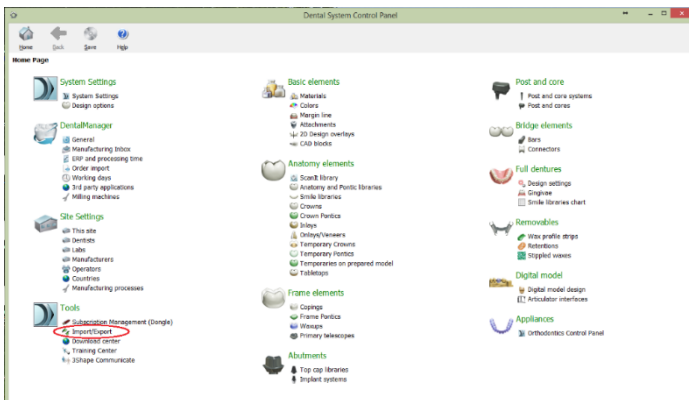
# Exocad DESS ABUTMENTS LIBRARY\_(Version)

## ENGLISH

- 1 • Copy the file "Exocad DESS ABUTMENTS LIBRARY\_(Version).7z" after the download and paste it in the index for the Exocad libraries: DentalCADApp\library\implant
- 2 • Unzip the file in the same index: DentalCADApp\library\implant
- 3 • Delete, if desired, the file "Exocad DESS ABUTMENTS LIBRARY\_(Version).7z" from your hard drive.
- 4 • The DESS Library is ready to be used.

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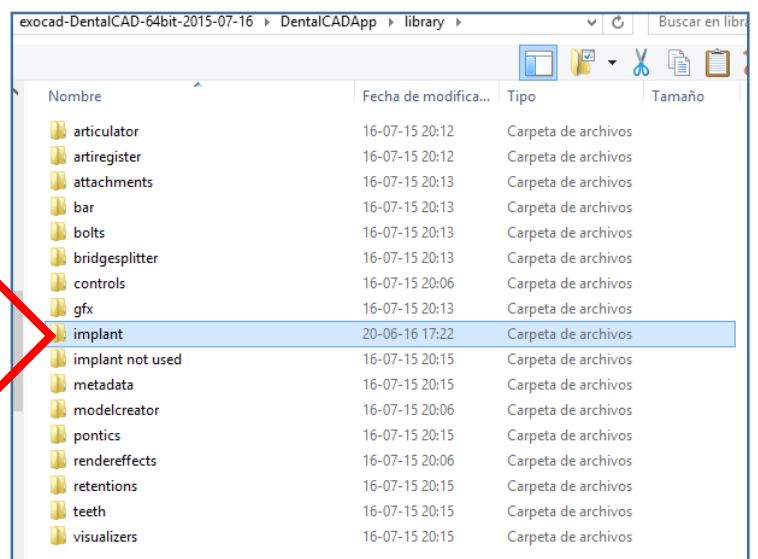
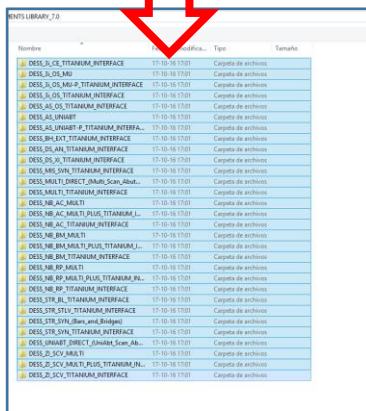
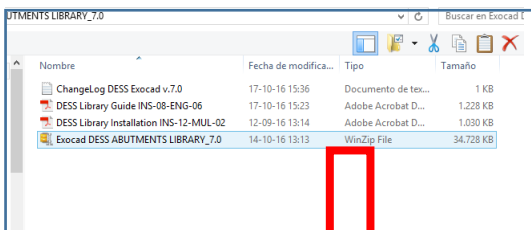
# Dental Wings DESS ABUTMENTS LIBRARY\_(Version)

## ENGLISH

- 1 • Start the program Dental Wings and choose the module "Implant Kits".
- 2 • On the chosen module "Implant Kit Editor" do a right mouse click. Choose with a left mouse click the menu item "Import Implant Manufacturer Kits".
- 3 • Choose the file "Dental Wings DESS ABUTMENTS LIBRARY\_(Version).zip". The loading of the folder can take a few minutes.
- 4 • The DESS Library is ready to be used.

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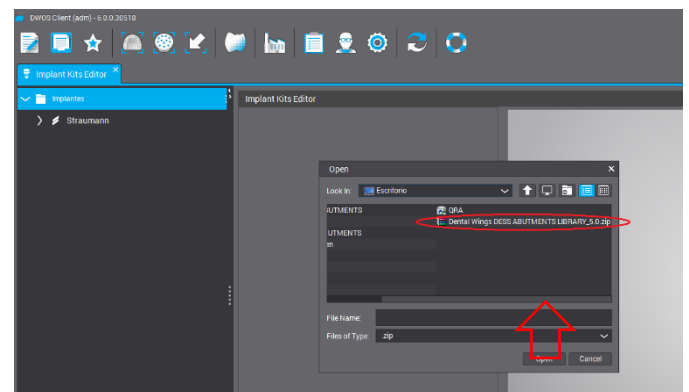
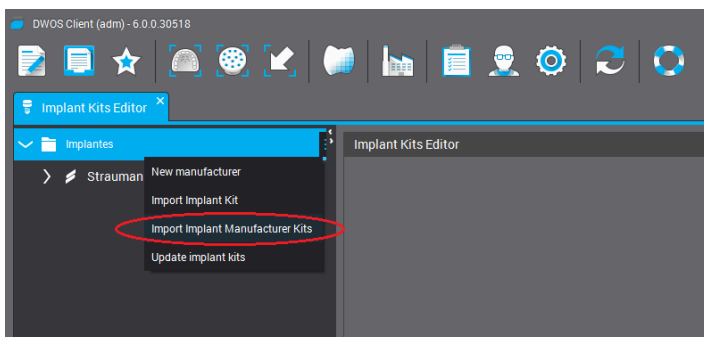
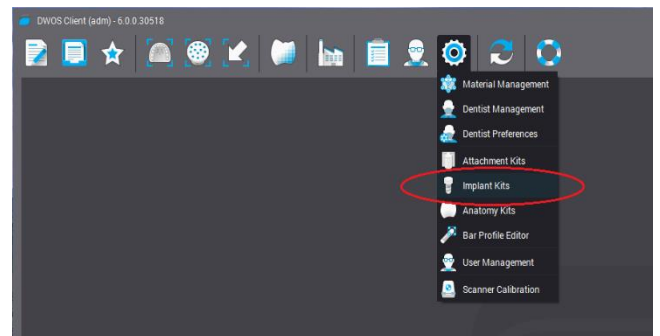
These instructions replace all previous editions. For any detailed information about DESS® products please contact your local distributor. The photographs of the products are for information purposes only. The type, reference and connection of each element should be checked. All DESS® products are manufactured according to ISO 9001:2008 and ISO 13485:2012 with CE marking for all products Class IIa. | 3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA | ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XIVE® and FRIADENT® and UniAbutment® are registered trademarks of DENTSPLY Implants | BioHorizons® registered trademark of BioHorizons, INC. | Brånemark System®, NobelReplace® and Replace Select™ and NobelActive™ and NobelReplace® CC and NobelSpeedy™ and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden | ITI®, Straumann® and synOcta® registered trademarks of Straumann Holding AG, Switzerland | MIS® registered trademark of MIS Implant Technologies Ltd | Zimmer Screw-Vent® and SwissPlus® registered trademarks of Zimmer Dental INC, USA | Dyna® registered trademark of Dyna Dental Engineering BV | Locator® registered trademark of Zest Anchors INC. | 3Shape® is a registered trademark of 3Shape A/S | EXOCAD® is a registered trademark of Exocad GmbH, Germany | Dental Wings® and DWOS® are registered trademarks of Dental Wings, Inc.